

Virucidal and skin compatible!

Neoseptin®

alcoholic hand disinfectant
disinfection of small surfaces and medical devices.

PRODUCT INFORMATION

Please read carefully!

NEOSEPTIN®

Ready for use! For professional use!

COMPOSITION

100 g solution contains: 79,9 g Ethanol 96%

RANGE OF APPLICATIONS

- Control of bacteria (tb included), fungus and viruses
- Disinfection of hands as well as of small surfaces and medical devices

MODE OF APPLICATION / DOSAGE

Use externally and **undiluted** only! Disinfection of small surfaces and medical products: Apply Neoseptin® and rub with a dry cloth. Surfaces and objects have to be completely wet with the product (max. 50 ml/m²)

| Hand disinfection according to the standard procedures of the DGHM of 2001/2002, under clean and dirty conditions | | |
|--|------|--|
| bactericidal, levurocidal | 30 s | |
| Tests according to the DVV/RKI directive | | |
| virucidal against enveloped viruses (incl. HIV, HBV, HCV) according to the recommendations of RKI 1/2004 | 30 s | |
| EN-norms (clean conditions) | | |
| Virucidal against enveloped viruses PLUS according to EN 14476 (adeno virus and MNV) | 30 s | |
| Virucidal according to EN 14476 | 30 s | |
| EN norms hygienic and surgical hand disinfection | | |
| EN 1500 | 30 s | |
| EN 12791 | 90 s | |
| Disinfection by wiping according to the DGHM standard procedures of 2001/2002, with mechanical effect, Clean and dirty conditions | | |
| Bactericidal, levuricidal | 1 mn | |

WARNINGS / STORAGE

- Do not apply on open wounds and on mucous membranes!
- Neoseptin® is easily inflammable.
- Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not smoke.
- Store this medical device in a dry, cool but frost-free place. Keep it away from ignition sources and out of reach of children.
- Do not use beyond expiry date.

PROPERTIES

- Contains no perfume
- Comprehensive spectrum of efficacy
- Reliable action in the presence of protein, serum and blood
- Well tolerated by the skin even with frequent use (moisturising and re-fatting properties)
- leaves the skin with a pleasant smooth feeling.

SPECIAL PRECAUTIONS

Neoseptin® is a medical device of class II A: CE 0482.

Use disinfectants safely. Always read the label and product information before use.

Certified and listed in the relevant reference documents

STANDARD PACKAGINGS / TRANSPORT UNITS

| | |
|--------------------------------------|------|
| 12 x 500 ml bottle | 2700 |
| 6 x 1 l liter bottle | 2705 |
| 15 ml Sanistick with spraying nozzle | 2708 |

Lysoform Dr. Hans Rosemann GmbH

Kaiser-Wilhelm-Str. 133, D – 12247 Berlin

Tel.: +49-30-77992-0 Telefax: +49-30-77992-219

www.lysoform.de, e-mail: kontakt@lysoform.de

1118-04